

Marisol Jet hypertonic spray for nose with sea water

Marisol Strong Jet hypertonic spray for nose with sea water

Power of the sea in spray

Adequate hygiene of the nose is important for babies, children and adults. Excess mucous in the nostrils can increase the spread of microbes and cause extra dryness of the mucous in the nose.

Marisol Jet / Marisol Strong Jet hypertonic spray for nose with sea water strongly hydrates nasal cavity thanks to the mechanical treatment and helps rinsing and elimination of the mucous, which establishes physiological function of the mucous membrane.

In **Marisol Jet / Marisol Strong Jet hypertonic spray for nose with sea water** key properties of natural sea water are preserved.

Marisol Jet / Marisol Strong Jet hypertonic spray for nose with sea water contains sea water from Cancale bay in Brittany, which is known for its water quality. Cancale bay has remarkable amplitude of high and low tide (12 m) and powerful sea currents, which prevents the contamination of sea water. As this simple combination of natural factors provides constant purification of the water, the result is constant purity. The sampling of the water is automatic, when the quality of the water is optimal (after high tide). The region where the sampling is done is classified as Class A and is constantly controlled for sanitary aspects (microbiology, chemical contamination, phytoplankton, pH)...

Ingredients: sea water, purified water.

Indications:

Marisol Jet hypertonic spray for nose with sea water is a medical device that can be used by children and babies older than 3 months.

Marisol Strong Jet hypertonic spray for nose with sea water is a medical device that can be used by adults and children older than 3 years of age. It is recommended for infections of nasopharyngeal mucosa, runny nose, infections of paranasal sinuses, as well as snoring caused by dry nose mucous. Hypertonic spray for nose with sea water removes excess fluid for mucous cells by its osmotic effect, providing mechanical cleaning and decongestion of upper airways, as well as reducing swelling of the mucous membranes.

Right use of **Marisol Jet / Marisol Strong Jet hypertonic spray for nose with sea water** in everyday nose hygiene helps in nasopharyngitis, obstructive rhinitis and sinusitis, irritation and nose dryness, as well as hoarseness caused by dryness of the mucous membranes. Also, everyday use is indicated as adjuvant therapy in the treatment of ear, nose, throat and pediatric illnesses.

Instructions for use:

1. Gently put the top of the applicator in the entry of the nose.
2. Push the spray pump.
3. Let the excess solution leak, so the mucous can be removed, and then blow your nose.
4. Repeat the procedure with the other nostril.
5. Clean the top of the applicator with water and dry it. Apply in each nostril as often and as long as it's needed.

Instructions for use in babies:

1. Put the baby in laying position with the head slightly tilted on one side, so that the solution could easily enter the nostril.
2. Disperse the spray by pressing the pump for several seconds.
3. Put the baby in upright position so the content from the nose could leak and pick it up with a tissue.
4. Repeat the procedure in the other nostril.
5. Gently wipe the region around the nose without dragging the tissue deep into the baby's nose. Clean the top of the applicator with water and dry it.

Apply in each nostril 2-3 times a day.

Warnings:

- Out of hygienic reasons, as well as to prevent infection, spray bottle should be used by one person only.
- After first opening, don't use the spray longer than 3 months.
- Keep out of reach of children.
- Don't use the product after the expiry date imprinted on the box and bottle.
- Keep on dry and cool place, on temperature below 40°C.
- Keep protected of direct sunlight and other heat sources.

Packaging:

Spray bottle under pressure, 120 ml with nose applicator.

- **Marisol Jet hypertonic spray for nose with sea water** – with delicate dispersion – suitable for children and babies older than 3 months.
- **Marisol Strong Jet hypertonic spray for nose with sea water** – with mild dispersion – suitable for adults and children above 3 years of age.

Authorized representative of the manufacturer and distributor: ESENSA d.o.o., Viline vode bb, Belgrade, Republic of Serbia, www.esensa.rs

PHARMALINK, S.L.

Av. Universitat Autònoma, 13 Parc Tecnològic de Vallès

08290 Cerdanyola de Vallès – BARCELONA (Spain)

www.pharmalink.es