

## **Oto Perfect aqua clean**

### **Ear spray solution, 30 ml**

Oto Perfect aqua clean ear spray solution is a hydrogen peroxide-based medical device. It is used to clean the ear canal, soften and remove ear wax, as well as a ceruminous plug.

In contact with the skin, hydrogen peroxide releases nascent oxygen, causing effervescence that mechanically fragments ear wax (which manifests as a "crack in the ear" and is an evidence of the effectiveness of the preparation). Additionally, liberated nascent oxygen from hydrogen peroxide acts antiseptically, contributing to the prevention of external ear infections. By using Oto Perfect aqua clean spray solution, the outer ear canal is cleaned, eliminating discomfort and reducing the risk of infection.

### **The use of Oto Perfect aqua clean spray solution is recommended:**

- To maintain ear hygiene
- In people with a natural tendency to over-produce cerumen
- To remove the ceruminous plug
- For people who wear hearing aids or often wear headphones
- To clean the ear just before administering the auricular medicines

### **How to use Oto Perfect aqua clean ear spray solution?**

Before first use, press the sprayer a couple of times from the bottle until a uniform dose of spray begins to flow. Warm the spray bottle in your hand (1 minute). Tilt the head to the side and inject a single dose of Oto Perfect aqua clean, making sure that the applicator rests only slightly on the earlobe. Leave for 5 minutes, keeping the head tilted laterally. Then straighten your head, let the solution drain and gently wipe your ear. In children, gently pull the earlobe backwards and then inject the product. To maintain ear hygiene and prevent the secretion of cerumen in the outer ear canal, apply the spray 2-3 times a week. People who wear hearing aids may repeat the procedure more often (several times a week). In case of ceruminous plug secretion, apply one dose of spray solution (in adults more if needed) and leave to act for 10 minutes, keeping head slightly tilted. Repeat as necessary.

### **Benefits of using Oto Perfect aqua clean ear spray solution:**

- Thanks to the spray form, the preparation can be applied in a standing or sitting position, allowing comfortable application
- The spray is dosed evenly into the ear canal
- No flushing of the ear is needed after administration

### **When not to apply Oto Perfect aqua clean for ears:**

- In case of perforation or lack of eardrum
- In case of ear canal injury
- In case of hypersensitivity to hydrogen peroxide or any of the ingredients of the product
- In case of an external body in the ear
- Before and after surgery
- For children under 3 years of age

**Warnings for the safe use of Oto Perfect aqua clean ear spray solution:**

Oto Perfect aqua clean is not a drug. In case of ear pain, seek medical advice.

Oto Perfect aqua clean can lead in some people to a slight irritation of the outer ear canal in the form of a transient sensation of slight burning, itching or redness of the skin. In the case of an allergic reaction to this product, discontinue use and seek medical advice.

**Advice:**

Mechanical cleaning of the auditory canal (such as cotton swabs) is not recommended as it can lead to micro-injuries that are easily infected when in contact with contaminated water or dust.

**How to store medical device Oto Perfect aqua clean ear spray solution:**

Store Oto Perfect aqua clean in its original packaging, in a dry and dark place, at a temperatures below 25°C. Keep the bottle upright, tightly closed, out of the reach of children.

**Shelf life: 2 years.**

Shelf life after first opening: 30 days.

**Packaging:** Folded cardboard box with plastic spray bottle containing 30 ml solution (approx. 220 doses) and instruction manual for medical device.

**Medical device Oto Perfect aqua clean ear spray solution contains:**

**Active substances:** 1 ml of spray solution contains 30 mg of stabilized hydrogen peroxide.  
Solution: purified water

**Other products from Oto collection:** Oto Perfect olive oil ear spray solution, Oto Perfect aqua acida ear spray solution, Oto CeruClean ear spray gel.

**Date of issue of the instruction manual of the medical device:** 10.06.2019.

**Number of decision on the registration of a medical device in the Register of Medical Devices:** 515-02-00865-19-003 from 08.08.2019.

**Manufacturer:**

ESENSA d.o.o., Viline vode bb, Belgrade, Republic of Serbia

**Distributor:**

FARMALOGIST d.o.o. Beograd, Mirijejski bulevar 3, phone no: +381 11 33 15 090