

Oto aqua clean

Spray solution for ears 30ml

Composition: Aqua, Hydrogen Peroxide stabilized.

Mode of action:

Hydrogen Peroxide in contact with water liberates nascent oxygen which has antiseptic effect.

It is used for washing ear canal, softening and removal of ceruminous plug.

Purpose of use:

- for maintaining ear hygiene
- in cases of excess cerumen formation
- in cases of ceruminous plug
- before using ear drops to wash ear canal and make better contact of medication and skin
- recommended for persons who carry ear plugs because they are prone to making ceruminous plug.

Instruction for use:

Before using the bottle with solution, warm it up in your hand (1 minute).

Tilt the head to the side and inject the appropriate dose of solution, taking care that to push the doser very gently. Leave 5 minutes with tilted head. Then straighten the head, let the excess leak out and gently wipe the liquid. Do not rinse. In children gently move the ear backwards and then apply the solution.

- for keeping ear hygiene apply one dose of spray solution and leave for 5 minutes with head tilted. Then straighten the head, let the excess leak out and gently wipe the liquid. Repeat this 2 or 3 times a day, and in persons that use earplugs even more times a week.

- when ceruminous plug is formed apply one dose of spray in the ear (in adults even more) and leave for 10 minutes keeping the head tilted. If needed repeat the procedure. Don't use earbuds to remove excess cerumen, just let the content leak out and gently wipe the ear. Do not rinse. During the use of the product you can feel crackling, which is the proof that the solution is effective.

Thanks to mechanical spray pump tiny droplets are formed that distribute evenly, thus making better therapeutic effect.

Caution:

This product **should not be used** in children under **3** years old.

Do not use this product if there is:

- a perforation or absence of eardrum
- hypersensitivity to hydrogen peroxide
- injury in the ear canal
- before and after surgery
- foreign body in the canal

In the case of the pain in the ear look for the doctor's advice.

Storage conditions:

Keep the product on temperatures below 25°C, protected of light.

Expiry date: 2 years. After opening the bottle, use the solution in 30 days.

Packaging: Box with a bottle with 30 ml of solution with dose sprayer

Manufacturer:

ESENSA d.o.o.

Slobodna zona Beograd, Viline vode bb

Republic of Serbia

www.esensa.rs