SRB BIH MNE

Baby &amp; Kids

# MARISOL®

## NAZALNI ASPIRATOR

### MARISOL BABY&KIDS NAZALNI ASPIRATOR

#### PREHLADA KOD DECE

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## EN MARISOL BABY & KIDS NASAL ASPIRATOR

### CHILDREN'S COLD

1. Colds are quite common for babies.
2. Babies cannot blow their noses themselves.
3. Blocked nose prevents normal breathing, drinking, and sleeping.
4. Consequential diseases of ears, sinuses or bronchi are possible.
5. Removal of secretion and keeping the airways clear is therefore especially important.

### EFFECT OF MARISOL BABY & KIDS NASAL ASPIRATOR

1. Marisol Baby & Kids Nasal Aspirator was specially developed for treatment at home.
2. Nasal secretion is completely and gently removed in a few seconds.
3. Baby can breathe, drink and sleep freely immediately.
4. Effect was proven in a study by the University Children's Clinic Vienna.
5. Marisol Baby & Kids Nasal Aspirator is suitable for children from birth.
6. Special construction reduces the suction strength to baby-friendly nose blowing (a safe pressure of 0,16 bar).
7. Marisol Baby & Kids Nasal Aspirator is compatible with every vacuum cleaner (possibly with adapter).


### ASSEMBLY & USAGE

1. Before first use, clean the product according to the CLEANING chapter.
  2. Place suction head A on part B.
  3. Connect part C to the vacuum cleaner.
  4. Switch on the vacuum cleaner, regulate suction if necessary.
  5. Hold the child's head as still as possible.
  6. Guide suction head A to the nostril.
  7. Suck for a few seconds until secretion is removed.
  8. Repeat the process on the 2nd nostril.
- Use small parts only under adult supervision.*

### CLEANING

1. Disassemble parts A and B.
2. Clean the suction head (A) under hot water with detergent and a brush.
3. Boil the suction head or sterilize it in a sterilizer.
4. Attention: No pressure sterilizer.
5. If dirty: rinse part B with detergent and hot water.
6. If dirty: rinse the hose with detergent and hot water.
7. Attention: Do not leave parts in the detergent.

### Easy to disassemble and clean

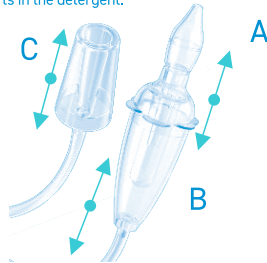### LEGAL NOTES

1. **Intended use:** The product is intended for the temporary, gentle removal of nasal secretions in infants/toddlers to facilitate breathing and promote well-being.
2. **Indication:** Removal of nasal secretions in infants/toddlers, especially in the case of colds, allergic reactions or other illnesses that lead to blocked nasal passages, as well as in infants/toddlers who have difficulty breathing, sleeping, or eating due to a blocked nose.
3. **Contraindications:**
  - \* Injuries or irritations in the child's nasal passages (e.g. fresh wounds, bleeding, inflammation)
  - \* Known allergies or hypersensitivities to materials in the product
  - \* Children with anatomical abnormalities or chronic respiratory problems for whom the use of a nasal aspirator is associated with risks. In such cases, medical advice should be sought before use.
4. **Intended users:** Marisol Baby & Kids Nasal Aspirator is intended for use by adults, typically parents or caregivers using the nasal aspirator on infants and young children. Users do not require any special medical training, but should read the instructions carefully and use the nasal aspirator according to the instructions to ensure safe and effective use.
5. Can be used with vacuum cleaners with a suction power of 800-2,000W.
6. Store the device at room temperature in a dry place. Protect it from direct sunlight and heat radiation.
7. **Study:** Application study of a technically identical nasal aspirator in May 2005 by the University Clinic for Paediatrics and Adolescent Medicine at the Medical University of Vienna.
8. All serious incidents that have occurred in connection with the product must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### WARNINGS

1. Only use under adult supervision: The nasal aspirator should only be used by adults. Children should not use this product on their own.
2. Be careful with sensitive nasal mucous membranes: Children with sensitive nasal mucous membranes or a tendency to nosebleeds should be careful to avoid irritation or injury.
3. Do not insert too deeply: The nasal aspirator should not be inserted too deeply into the nostrils to avoid the risk of irritation or injury to the nasal mucous membranes.
4. Do not use if there are signs of infection: If there are visible signs of a nasal or respiratory tract infection (e.g. severe mucous membrane irritation, severe nosebleeds, or purulent discharge), you should consult a doctor before using the aspirator.
5. Regular cleaning required: Clean the nasal aspirator thoroughly before and after each use according to the instructions to avoid infection.

 Reshape Healthcare Würzburggasse 16 A-1130 Vienna  
office@airmedsvac.net