EN

Oto Prevent MultiClean spray is a multi-purpose product for maintaining ear hygiene, but also serves as a product for cleaning and disinfecting earbuds for mobile phones, bluetooth headsets, hearing aids and the like.

Formulated on the basis of hydrogen peroxide and tea tree oil, Oto Prevent MultiClean Spray provides reliable protection against bacteria and fungi and reduces the risk of infection and inflammation of the ear.

How is Oto Prevent MultiClean spray used?

1. To maintain ear hygiene:

Before using for the first time, press on the pump several times until a uniform dose of spray begins to emerge from the bottle. Warm the spray bottle by holding it in your hand (1 minute). Tilt your head to the side and spray a single dose of Oto Prevent MultiClean spray into your ear, making sure that the applicator rests slightly on the earlobe. Allow the product to exhibit its effect for a few minutes, keeping your head slightly tilted. Afterwards, straighten your head, let the solution drain away by leaking out and then gently wipe your ear with a dry cloth. Spray Oto Prevent MultiClean into your ear 2 - 3 times a week. If necessary, the process can be repeated more often.

Notes: Do not use the product in case of perforation or lack of eardrum, ear canal injury, hypersensitivity to hydrogen peroxide, tea tree oil or any of the ingredients in the product.

Do not use if there is a foreign body in the ear, before or after surgery nor on children under three years of age.

2. To clean and disinfect earbuds / hearing aids:

1. Remove any spongy, silicone, rubber or memory foam parts from earbuds / hearing aids (if any), wash them with lukewarm water and soap and allow them to dry.

2. Remove any visible dirt from the earbuds / hearing aids first - wipe them with a dry soft cloth or tissue paper.

3. Spray Oto Prevent MultiClean spray on a clean, dry soft cloth or tissue paper and gently wipe the surface of the headphone / hearing aid.

Do not spray the product directly on earbuds or hearing aids.

4. When cleaning the earbuds, spray Oto Prevent MultiClean on a soft-fiber brush (such as a soft toothbrush), shake off the excess solution, and clean the headset by keeping the brush perpendicular to the surface you are cleaning. To avoid liquid or dirt entering the earbuds themselves, always keep them in the position with the speakers facing down and clean by using strokes of the brush that are in one direction only.

Use a specialized hearing aid brush to clean your hearing aids.

5. Allow the earbuds to dry for about 12 hours with the speakers facing down, in a dry and dark place (avoid damp areas such as bathrooms).

Storage conditions: Store in original packaging, in a dry and dark place, at a temperature not exceeding 25 °C.

Keep the bottle upright, tightly closed, and out of the reach of children.

Expiry date: 2 years. The date of manufacture and serial number are printed on the box. After first opening, use within 30 days.

Expiry date after first opening: 30 days.

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